GB **English – instructions**
Mediroyal Proxi Thumbs

General information - Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the backside of the white textile label.

Indications - The product has been design to provide support to the CMC- and MP-joints of the thumb.

Contraindications - Patients with instable fractures should not use this brace.

Pre Application - To select the right size, take the circumferential measurement around the wrist. Use the measurement chart to determine the most suitable size. Note that the product made in left or right versions.

Adjustment Proxi Light - The Proxi Light thumb has an external reinforcement that provides stability to the thumb. This film will adjust to the patient by itself over time. The outer design can be trimmed with a pair of scissors if needed. Avoid cutting in the area of the hook and loop closures.

Adjustment Proxi Plus/Proxi DeQuervain - The Proxi Plus/Proxi DeQuervain thumb has an internal thermoplastic reinforcement that has been laminated with the external reinforcement film. The shape is pre-moulded and can be fitted immediately. If the volume is too big, simply fold the thumb part and press it together carefully. That changes the thermoplastic and makes the size smaller. If you want to make the size bigger simply open it up and hold for a few seconds. The outer design can be trimmed with a pair of scissors if needed. Avoid cutting in the area of the thermoplastic and the hook and loop closures.

Application - Close the thumb grip of the brace before application. Apply the thumb through the opening. Apply the strap around the wrist and re-adjust the thumb strap if needed.

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
Removal - Open the Velcro straps and remove the brace. Close all straps when the brace is not in use.

Material content - Check the material content on the white label.

Caring instructions and precautions - We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag. Never wash in temperature over 40 degrees Celsius.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the backside of the label on the cover.

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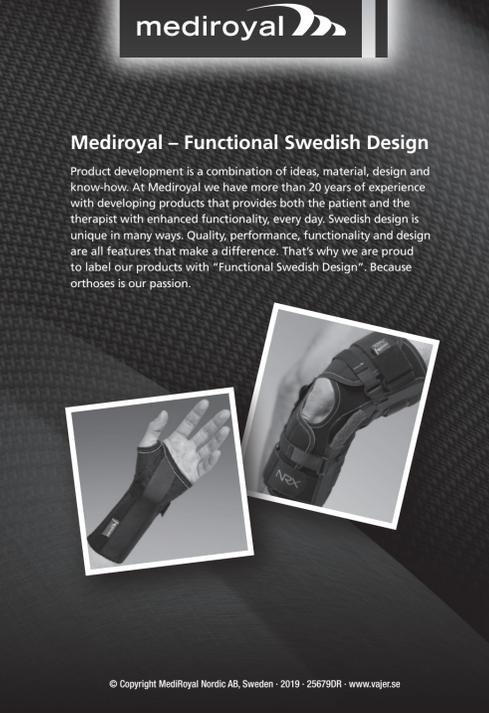
Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, every day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses is our passion.

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