English – instructions

RESTO® Hand Splint Functional Resting Position

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the white textile label on the product.

Indications · The product has been design to provide a functional support to the fingers, thumb & wrist during long term immobilization. It can be used for patients with edema to provide a correct position. It can be used by patients with inflammatory processes to provide a resting position to decrease pain.

Contraindications · Patients with high spasticity will require a more stable solution. Consider the Mediroyal TONUS®/DYFEX® products instead.

Pre Application · To select the right size, take the width measurement over the MCP-joints then use the measurement chart.

Adjustment · Evaluate the wrist and fingers. Adjust the frame by using your hands. Start with the finger platform, then the thumb and last the wrist. Do not extend the thumb piece into abduction.

Application · Open the Velcro straps. Position the fingers, thumb and wrist into the splint. Start by closing the straps around the wrist to secure the brace. Then attach the strap over the fingers. On E26 there is a modular strap that can be applied separately ulnar-radial or radial-ulnar to provide more support in flexion. Make sure that the straps are not too tight as that might cause edema.

Position · The RESTO splint is made from a malleable aluminium frame and platforms. This will allow you to gradually adjust the splint depending on the progress of the patient. Avoid bending repetitive back and forth over the same place as this might cause fatigue damages to the structure.

Removal of the splint · Open the Velcro straps and remove the splint.

Lining · The E25 has a Neptune™ material on the surface that provides comfort and moisture transportation. On the E26 there is a self attaching lining made from Neptune™ and ARX self attaching material. This means that the lining can be machine washed separately from the splint to maintain an optimal hygiene. The E26 linings can be bought separately as replacement linings if needed.

Caring instructions and precautions · We recommend that the cover should be washed once a week. Follow the washing instructions on the white label and always use the laundry bag. Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on white textile label.

For more information consult www.mediroyal.se

E25 RESTO® Basic

E25 RESTO® Basic & E26 RESTO® PLUS

Size chart

Size mm	E25 RESTO® Basic				MCP width in mm
	XS 45–65	S 60–75	M 70–85	L 85–95	

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© Copyright MediRoyal Nordic AB, Sweden · 2020 · 25732DR · www.vajer.se