mediroyal 

TONUS Hand Roll

TONUS 

mediroyal 

CE

Size Chart

Universal size
Left and Right version

English	8	GB
Svenska	9	SE
Suomi	10	FI
Deutsch	11	DE
Français	13	FR
Italiano	14	IT
Español	15	ES
Português	17	PT
Nederlands	18	NL
Eesti	19	ET
Lietuvos	20	LT
Latvijas	22	LV
Slovenski	23	SL
Polski	24	PL
Русский	25	RU
Chinese Traditional	27	CH
Chinese Mandarin	28	CM
Japanese	30	JP
Korean	32	KO
Malay	33	MY
Arabic	35	AR
Persian	37	PE

1

2

3

4

TON813 · Finger Separating Strap

Instruction

Mediroyal Tonus Hand Roll

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the textile label.

Indications · The product has been designed to open the fingers and to reduce pressure to the palmar area for patients with spasticity.

Contraindications · Patients with open pressure wounds in the palmar area should not use the product unless the wound has been sealed with a wound dressing.

Pre Application · To select the right size, take the width over the MCP-joints. If the patient has a very high tonus, it might be beneficial to try to inhibit the spasticity before application. Correct positioning of the shoulder, external rotation of the upper limb and a gentle extension and stretch of the wrist and fingers might make application easier.

In order to insert the air roll into the palmar area you could try to flex the wrist in order to release tension over the flexor tendons to open the fingers more. Make sure that the thumb will not hyper-extend.

Application · The valve should be placed on the ulnar side. Deflate the bladder completely by using the back tip of the pump. Position the roll into the palmar area and let the fingers grip the roll. You can secure the strap around the wrist by crossing it or wait until you have fully inflated the bladder. The strap will prevent the roll from slipping out of the hand if spasticity decreases.

Insert the clear tip of the pump into the valve. Inflate the bladder until the finger tips can't reach the palmar area. Sometimes it's easier to fill the bladder to maximum and then slightly deflate it in order to make the roll soft and comfortable.

The hand roll can be used both day- and nighttime. The spasticity reduction effect in the hand roll is limited since the wrist is not stable. It's possible to combine the hand roll with a normal wrist brace in order to prevent wrist flexion.

Finger separation · If there is a need to separate the fingers, the TON813 Finger Separation Strap can be used. Apply the finger loops first. Then apply the hand roll, secure it around the wrist and inflate. Then run the finger separation strap around the wrist and secure.

Removal · To remove the splint, start by deflating the bladder with the black tip of the pump. Then open the strap around the wrist

Material content · If the patient is allergic, check the material content on the label cover before use.

Caring instructions for the cover · The cover is made from a special composition of wool fibers that enhance ventilation and moisture transportation and also provides pressure absorption. The cover can be machine washed in 60 degrees C using the enclosed laundry bag. The cover should not be tumbled, it should air dry. Wool is a natural fiber; therefore the cover might release lint or fibers during the first washings. Once the cover has dried after washing, you can use a hair brush to make the surface soft again. We recommend that the cover should be washed at least once a week. Replacement covers can be ordered separately.

Caring instructions and precautions · The bladder can be wiped with a mild soap solution or surface alcohol for disinfection. We do not recommend washing the bladder along with the cover in a washing machine.

Patients with sensitive skin, diabetes or poor circulation should be observed frequently for pressure marks. Patients with allergies should check the material content on the label before use. For more information consult www.mediroyal.se

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mediroyal 

Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, very day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses are our passion.

Mediroyal Nordic AB · Box 7052 · Staffans väg 6 B · SE-192 07 Sollentuna · Sweden
Tel +46 8 506 766 00 · info@mediroyal.se · www.mediroyal.se

© Copyright MediRoyal Nordic AB, Sweden · 2019 · 25579DR · www.vajer.se